
INSTRUCTIONS FOR USE

Contents

English	4
Deutsch	6
Nederlands	8
Dansk	10
Norsk	12
Svenska	14
Français	16
Italiano	18
Español	20
Português	22
Româna	24
Ελληνικά	26
Türkçe	28
Suomi	30
Eesti	32
Magyar	34
Latviešu	36
Lietuvių	38
Polski	40
Česky	42
Slovenčina	44
Slovenščina	46
Български	48
Hrvatski	50
Íslenska	52


Relax Wheelchair Cushions and Seating Solutions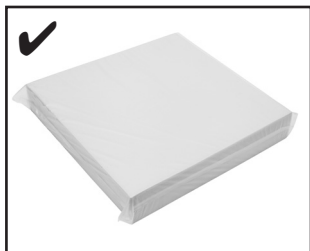


Trulife Product Codes

Product Code	Cushion Description
RT001	Trio with Black Outer Cover
RD003	Duogel with Black Outer Cover
RD006	Duogel with Wipeable Cover
RDP003	Duo-Profile with Black Outer Cover
RDP006	Duo-Profile with Wipeable Outer Cover
RDP009	Duo- Profile with Spacer Fabric Cover
RG004	Gelcell with Black Outer Cover
RG002	Gelcell with Spacer Fabric Cover
RG005	Gelcell G2 with Black Outer Cover
RC002	CoolCell with Spacer Fabric Cover
RC004	CoolCell with Black Fabric Cover
RE005	Easy with Black Wipeable Outer Cover
REH005	Easy High Profile with Black Wipeable Outer Cover
RG003	Gelcell G2 with Spacer Fabric Cover

Duogel/Duo-Profile/Easy/Easy High/Trio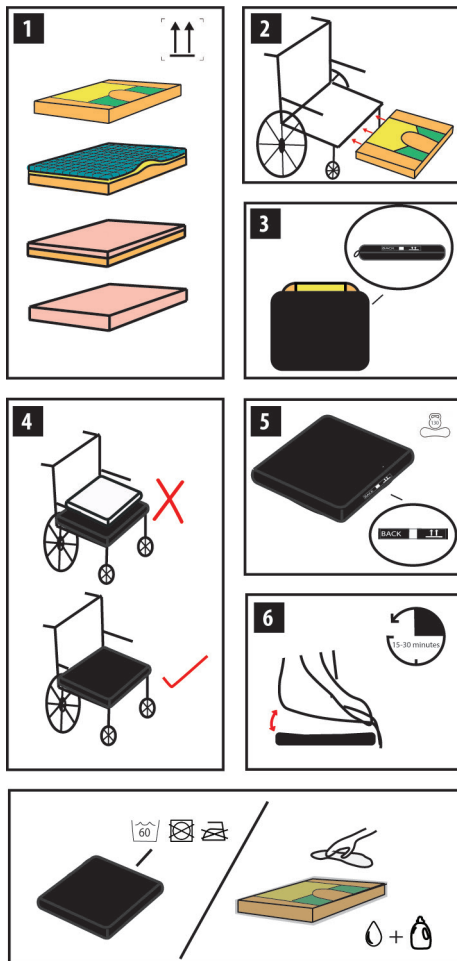

Gelcell/Gelcell G2/Coolcell

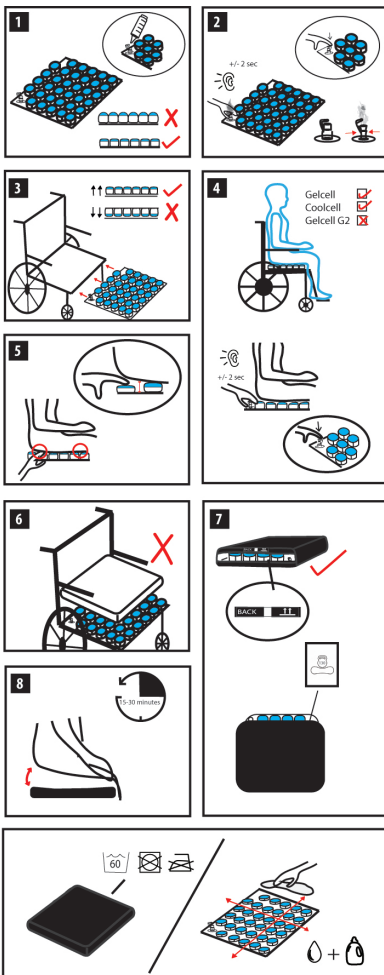Use of your RELAX Cushion

The RELAX range of cushions are designed to provide support, weight distribution and pressure distribution for wheelchair users and people of restricted mobility who spend extended periods of time in a seated position.

Trulife Relax range is designed to help aid with weight and pressure distribution and sitting comfort. However, no cushion can completely eliminate sitting pressure or prevent skin breakdown. The Relax cushion is not a substitute for good skincare including, proper diet, cleanliness, and regular weight shifts/ pressure reliefs. In addition, prior to prolonged sitting, every cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing.

- Each RELAX cushion is recommended to be used with the cover provided.
- Ensure the back of the cushion, where marked, is placed to the back of the chair and the top surface, where marked, is facing upwards as indicated on the label at the opening.
- Avoid using additional padding or covers as this could reduce the effectiveness of the cushion.
- The inner cushion should be inspected regularly – if damaged it should not be used.
- The cover if damaged should not be used.
- When fitting the cushion cover ensure the back of the cushion is positioned to the back of the cover.
- It is very important to reposition the user on a regular basis.
- If your RELAX wheelchair and seating cushion has been subjected to extreme temperatures (-20°C/-4°F or +40°C/104°F) please allow to revert to room temperature before use.
- It is important to clean the cushion and cover between users to avoid cross contamination.
- RELAX wheelchair and seating cushions are guaranteed against manufacturing defects for 2 years from date of manufacture.
- Weight limits are indicated on the Relax wheelchair and seating cushion cover.
- It is important to note that pressure relief will be compromised as weight increases.
- Please store all products flat.
- Avoid placing heavy or sharp objects on any of the products..

Disposal instructions

- Any disposal practice must be in compliance with all local and national laws and regulations. All packaging materials used for your RELAX cushion can be recycled.
- If returning the product, the cover should be washed and the inner cushion disinfected before being placed in a sealed plastic bag.
- If you have any questions or comments regarding the RELAX range, please contact your local distributor.

Guidelines for inflating the Relax Gelcell/ Gelcell G2/Coolcell

- 1) Inflate the Gelcell/Coolcell cushion with the pump provided until the cushion is level, do not continue pumping, as excessive inflation will damage the cushion. (Refer to Diagram Page 3.)
- 2) Ensure the cushion is positioned with the valve to the back of the chair. Before placing the user on the cushion, squeeze the valve to release excess air – i.e., until air pressure is equalized inside and outside the cushion.
- 3) Close the valve.
- 4) The user now sits on the cushion in an upright position, ensuring the back is straight and maintaining as near as possible an angle of 90 degrees at the hips and knees.
- 5) Open the valve and squeeze to release air for approximately 2 seconds, or until it is felt by the user to have reached the most comfortable pressure. As air is released the cushion will conform to the shape of the user.
- 6) Close the valve.
- 7) Place your hand under the area of most bony prominence to ensure there is enough inflation between the user and base of cushion.
- 8) The cushion is now properly adjusted.
- 9) The level of inflation should be checked on a regular basis .

IMPORTANT NOTES

- Always ensure that the cushion is properly inflated and that the user is being supported by the cushion – i.e. the user never touches the base of the cushion.
- If the Gelcell/GelcellG2/Coolcell is being used by various users, follow the steps above each time.
- It is important to use the correct cushion cover with your Relax cushion.
- Please keep these instructions in a safe place for referral.
- If user shows signs of skin redness while using cushion discontinue use.
- Serious Incidents that occur in relation to these products should be reported to Trulife at quality_department@trulife.com and the Competent Authority of your member state.

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